# MEDISANA®

**DE Luftbefeuchter** AH 662

**GB Air humidifier** AH 662

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**IT Umidificatore** AH 662

**ES Humidificador** AH 662

**PT Humidificador** AH 662

**NL Luchtbevochtiger** AH 662

**FI Ilmankostutin** AH 662

**SE Luftfuktare** AH 662

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## IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

### *Explanation of symbols*

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

#### **WARNING**

These warning notes must be observed to prevent any injury to the user.

#### **CAUTION**

These notes must be observed to prevent any damage to the device.

#### **NOTE**

These notes give you useful additional information on the installation or operation.

Protection category II

**LOT**

Lot number

Manufacturer

### *Power supply*

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only connect the mains plug to the wall socket when the device is switched off.
- Never carry, pull or turn the device by the electricity cable and do not squeeze the cable.
- Position the mains cable so as to avoid the danger of falling.
- Do not touch the mains plug when you are standing in water and always take hold of the plug with dry hands.

### *For special people*

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- In the event of an allergic respiratory problem, consult your doctor before using the appliance.
- Should you experience medical complaints during use talk to your doctor. Should this be the case, stop using the device immediately.

### *Operating the device*

- The device is not determined for commercial use.
- Only use the device for its intended use as in the instruction manual. If misused, all guarantee rights shall become null and void.
- Do not use the device if it or the power supply cable is damaged, if it does not work properly or if it has been dropped or fallen into water.
- Keep the device and its power cable away from hot surfaces.
- If the power supply cable is damaged, it may only be replaced by **MEDISANA**, an authorised dealer or by qualified personnel. In order to prevent hazards, always send the unit in to the service centre.
- Place the device on a firm floor area when in use so that it cannot tip over.
- Do not take hold of a device if it falls into water. Unplug the mains cable immediately.

- Do not use the device if the water tank is empty or does not contain enough liquid.
- Always make sure that the device is switched off and unplugged from the socket when transporting, filling and emptying the water tank.
- Be careful of hot steam emissions.
- Switch off all functions after use and remove the plug from the mains supply.

## *Maintenance and cleaning*

- You may only clean the device yourself. If the device malfunctions do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the **MEDISANA** service.
- Cleaning and user maintenance shall not be made by children without supervision.
- Do not scratch the ultrasonic transducer when cleaning.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.
- Store the unit in a dry place.

## 2 Useful Information

Thank you for your confidence in us and congratulations on your purchase!

You have acquired a quality product with your purchase of the **MEDISANA** personal humidifier **AH 662**. To ensure the best results and long-term satisfaction with your **MEDISANA** personal humidifier **AH 662**, we recommend that you read the following operating and maintenance instructions carefully.

### 2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA** personal humidifier **AH 662**
- 1 Instruction manual
- 1 Cleaning brush

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your supplier without delay.

#### **WARNING**

**Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!**

### 2.2 Intensive personal humidifier

Very dry air increases susceptibility to infection and respiratory diseases and leads to fatigue and lack of concentration. Healthy air should have a relative atmospheric humidity of 40 to 55 %. The quiet,

energy-saving ultrasound technology of the personal humidifier uses oscillation to convert water into a fine mist that increases the moisture level of the air. This improves the air quality and prevents the mucous membranes and skin from drying out. The capacity of the device is suitable for a room with a maximum area of 8 m<sup>2</sup>. Pay special attention to the notes on cleaning and care.

**CAUTION** Over-intensive operation can result in saturating the room with moisture, which can affect the walls, windows and objects in the room. Therefore, use a hygrometer at regular intervals to check that the level of relative air humidity does not exceed 55%

## 3 Operating

**CAUTION** Make sure that the MEDISANA air humidifier is operated on a surface which is resistant to water, and that it cannot tip over. In particular, when operated with mineralised water, condensation / residues may be produced. Make sure that the mist being issued is not directed towards electronic appliances, furniture or walls.

### 3.1 Filling and emptying the water tank

Before you switch on the device for the first time, leave it for half an hour at room temperature. Detach the water tank **3** from the device and turn it over. Unscrew the cap **4**. Rinse out the water tank and then fill it up with a maximum of 0.9 l drinking water. Seal the tank again by screwing the cap **4** back in. Make sure that the water canister **5** is empty. Turn the water tank over again and place it back in the device so that it audibly engages. Do not move the unit when the water chamber is full. Dispose of the leftover water after use and rinse out the tank with fresh drinking water. Before using again, fill the tank with fresh drinking water. When you have switched off the unit and do not intend to use it for an extended period of time, remove and empty the water tank.

**CAUTION** Only use fresh drinking water (max. 40°C) in the device. You can also use demineralised water, if the water is very hard and calciferous. Clean the device at regular intervals and remove lime and other deposits, to maintain its efficiency. Do not pour the aroma oil directly into the water tank.

### 3.2 Switching the unit on and off

When required, remove the aroma compartment lid **1**, add a few drops of aroma oil to the compartment and replace the aroma compartment lid. Plug the device into a properly installed power socket. Turn on the humidifier by pressing the ON/OFF button **6** once. The integrated control LEDs in the lower section of the housing **8** light up blue. When the water tank is getting low, they flash blue and red alternately. When the water tank is empty, they light up red.

After a few seconds the mist from the outlet **2** will be visible. Switch off the device when not in use or if there is no water in the tank (the control LEDs light up red) by pressing the ON/OFF button **6** once. The device switches itself off automatically when the water tank is empty. With a full water tank, the maximum operating time is approx. 15 hours.

### 3.3 Faults and Fault Remedying

- *Device will not switch on (LED not illuminating):* Plug in the plug; Check power circuit, socket and fuse.
- *No mist (LED lights up red):* Fill water tank.
- *No mist although tank full:* Close the lid properly or too much water in the water canister.
- *Mist intensity too weak:* too much residue in the water canister, clean the ultrasonic transducer/water too dirty, replace.
- *White deposits:* Use clean water or at least 1/2 distilled water / 1/2 fresh tap water.

- *Unusual smell*: polluted or stagnant water, clean the tank and fill up with fresh water.
- *Unusual noise*: insufficient water in the tank, fill up with water/device is not on a solid surface, place on a stable and level surface.

If a fault cannot be remedied using the described method, please contact your service centre.

## 4 Miscellaneous

### 4.1 Cleaning and maintenance

- During everyday use it is advisable to give the device a thorough cleaning at least every three days. Also clean device before and after long storage periods.
- Before cleaning the device, make sure that it is turned off and the mains plug is removed from the socket.
- Remove the water tank from the device and empty it.
- Clean the outside of the unit with a soft, damp cloth.
- Please do not use any aggressive substances for cleaning, such as scouring cream or other detergents, as these may damage the surfaces and reduce the atomisation effect.
- You can rinse out the water tank with hot water (>65 °C) or an alcohol of at least 75 % vol. The atomiser chamber should be cleaned using an alcoholsoaked cloth and cotton bud. Take care to ensure that no water gets inside the humidifier.
- The ultrasound generator is the most sensitive component in the device. It is important for it to be kept clean and handled with care. Use the cleaning brush supplied to remove all the residue and dirt particles. Never clean the ultrasound generator using metal objects.
- If you do not intend to use the unit for an extended period of time, empty any remaining water from the tank and dry off the unit.
- Store the unit in a cool, dry place, preferably in its original packaging.
- Remove cap from underside of the water tank and store separately to prevent sleeve gasket from sticking.

### 4.2 Descaling

When using hard water, or water with a high mineral content, the device must be regularly descaled. Please use only citric acid for descaling (powder preparation from the chemist). (No liquid descaler! No descaling tablets! No vinegar!).

- Empty out the device completely (tank and lower section). Ensure that no liquid runs into the air outlet of the lower section.
- Place 1 tablespoon (approx.25g) of pure citric acid in the tank and fill up with 500ml of water at 60 °C. Screw the tank shut and gently shake until the citric acid powder has dissolved.
- Place the tank on the lower section. **DO NOT SWITCH ON THE DEVICE!**
- Allow the descaling solution to cool in the tank/device (at least 3 hours).
- Remove the tank from the device and, with a suitable brush (bottle brush or similar), scrub the interior and the mist chamber. Empty the contents of the tank and rinse well.
- Empty the lower section. Make sure no liquid gets into the air outlet. Rub off persistent lime scale using the brush supplied, a cloth or cotton buds. Take care around the ultrasonic transducer!
- Rinse the lower section well (Ensure that no liquid runs into the air outlet of the lower section).

### 4.3 Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your local authority or your supplier for information about disposal.

#### 4.4 Technical specifications

Name and model	: <b>MEDISANA</b> personal humidifier <b>AH 662</b>
Power supply unit	: 100 - 240 V~ 50/60Hz 0.5 A
Output	: approx. 12 W
Tank contents	: 900 ml
Sound emission	: approx. 35 dBA
Atomizer capacity	: maximum approx. 60 ml/h
Duration	: maximum of approx. 15 h
Dimensions	: approx. 15 x 15 x 22 cm
Weight	: approx. 0.45 kg
Connection cable length	: approx. 150 cm
Storage conditions	: clean and dry
Item number	: 60077
EAN number	: 40 15588 60077 7

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

#### 4.5 Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**MEDISANA AG**

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41468 NEUSS

GERMANY

E-Mail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the attached leaflet.

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