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Toilet Seat Raiser RFM with or without lid

available heights: 5 cm, 10 cm, 15 cm

Setting up

- Mount the L-shaped clamps as follows
- Place the seat raiser upside down on a flat surface and place the clamps in the receiving points. Slowly push them inside and twist the screw clockwise until the clamp catches.
- Adjust the clamps so that there is still place between clamp and toilet rim.
- Place the seat raiser on the toilet bowl so that the front lip touches the inner front rim of the bowl.
- Twist the screws clockwise until you feel a noticeable resistance and the the seat raiser will not budge on the bowl.
- **Attention!!:** The screws are designed as to turn easily. Do not turn over any further when you note a resistance that indicates the crews are already tight.
- Check that the Toilet Seat Raiser RFM won't move before using it.

Cleaning

- All parts can be cleaned with warm water and neutral detergents. Alternatively the seat raiser can be immersed in water and/or autoclaved for 1 minute at 85°C (except the clamps).

Warning

- Do not exceed the maximum load: 200 kg.
- Check that the Toilet Seat Raiser RFM won't move before using it.
- Do not use the seat raiser for a handhold while sitting down or getting up.

Attention: Do not overturn the screws!

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Rehaforum Medical GmbH