
# DR. VAC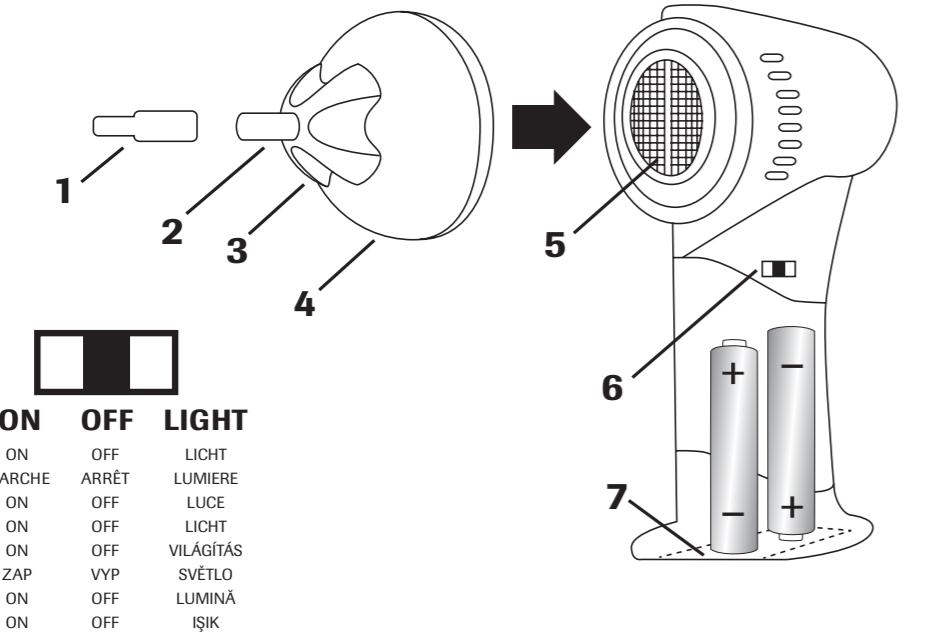

## EN INSTRUCTIONS FOR USE

### Operation

- Remove all packaging material and dispose of it in accordance with the valid legal provisions.
  - Open the battery compartment by sliding open and unfolding the battery cover.
  - Ensure the correct polarity (+ / -) and insert two new 1.5 V batteries, type AA or remove the dead batteries beforehand.
  - Close the battery compartment by shutting the cover and sliding it back.
- NOTES:** Requires 2 x 1.5 V batteries AA - not included. When the power of the device drops, always replace both batteries.
- Push the ON/OFF/LIGHT switch to LIGHT. With the light you can shine into your ear before use to see where deposits can be found.
  - Stick the silicon fixture on the nozzle of the ear cleaner.
  - Slide the ON/OFF/LIGHT switch to ON and insert the nozzle carefully into the ear to be cleaned.
  - Move the device carefully in circling forward and backwards movements to reach all deposits.
  - After use, pull the nozzle out of your ear again and put the ON/OFF/LIGHT switch to OFF.
  - Clean the fixture and collection tank thoroughly. The same needs to be performed if the fixture used is blocked during the cleaning.
- TIP:** An application after taking a bath or shower is recommended, as deposits are more easily removed then. For reasons of hygiene, a fixture should be used by one person only, and it should also be replaced at regular intervals.

### Cleaning and storage

- NOTE:** To clean the device and the accessories do not use any caustic or corrosive cleaning agents.
- Switch the device off - ON/OFF/LIGHT switch in the OFF position.
  - Pull the silicon fixture from the nozzle and clean the fixtures used with hot water and with a little soap, if necessary.
  - Turn the container tank off the device and clean it with hot water and with a little soap, if necessary.
  - Remove the filter and clean it with the cleaning brush. If necessary, the filter can also be cleaned with hot water and with a little soap, if necessary.
  - You can wipe the housing with a damp cloth and then wipe it dry with a towel.
  - The device and the accessories need to be fully dry before you put the parts back together.
  - Keep in a clean, dry place out of the reach of children and animals.

**Safety**  
Read the instructions for use carefully before using the device. Follow all safety notes to prevent damage from incorrect use. Keep the instructions for use for later reference. If the device is given to a third party, these instructions for use must also be handed over, if applicable. Dr. Vac is not a medical device and not suitable for medical use. Purpose and intended use: wellness product.

**EN**

**Scope of delivery**  
1 x Dr. Vac device  
8 x silicon fixtures (2 x yellow, 2 x green, 2 x red, 2 x blue)  
1 x cleaning brush  
1 x instructions for use

**Operating elements**  
1 - silicon fixture  
2 - nozzle  
3 - safety fixture  
4 - collection tank  
5 - filter  
6 - ON/OFF/LIGHT switch  
7 - battery compartment  
8 - light

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