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## Instructions for Use - Ear cleaner

### Dear Client,

Congratulations for choosing one of our quality products. Please follow these instructions carefully and keep them with you.

### Product description

Excess ear wax is removed from inside the ear by the gentle suction of the device, preventing blockages and infections. Thanks to the silicone attachments included, adult's and children's ears can be cleaned hygienically and safely. Can also be used for sensitive ears. Small and handy, easy to use.

### Taking into operation

1. Push the cover of the battery compartment downwards (see fig.1) and insert two AA batteries, observing the polarity as marked in the battery compartment. Close the battery compartment cover again.
2. Push one of the four silicone attachments onto the ear cleaner (see fig. 2). Exchange the silicone attachment each time after cleaning the ear. To clean the silicone attachment, simply rinse it out with water.
3. Now switch on the ear cleaner (ON). Do not insert the device too deep into the ear. Excess ear wax will now be removed from the inside of the ear by the gentle suction of the device. If the process is complete, switch the device OFF (see fig. 3).

### Cleaning

4. To clean the device, turn the collecting vessel in the direction of the arrow (see fig. 4) and remove it.
5. Clean the collecting vessel and the extractor with the small brush provided (see fig. 5).

### Warning notice

Improper use can lead to injury to persons or damage to property or the product itself.

- Keep out of reach of children.
- Rinse out the silicone attachment with sufficient water after each use.
- Remove the batteries when not in use.

### Technical data

Supply voltage: 2 x 1.5 V AA batteries, weight: 99 g

### Attention: Important battery notice

- We recommend to use alkaline batteries for best possible performance.
- Never mix old and new batteries or one-way batteries and accumulators
- Never use different types of batteries (eg alkaline and zinc carbon) at the same time.
- Remove the batteries if the device is not used for a long time. Leakage of batteries can cause permanent damage to the device.

### Instructions for battery disposal:

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Batteries are subject to legal provisions. After use they must therefore be returned to the point of sale or to a collecting point of public facilities for harmful substances. Batteries may never be disposed as household waste, in the yellow sack or other. Dispose of batteries only when completely discharged into a waste batteries collecting container and take precautions against short circuit (eg by masking the poles).

### Safety instructions

**Important:** Damage caused by not adhering to the operating instructions is not covered by guarantee. Maximex GmbH & Co. KG accepts no liability whatsoever for any resulting consequent damages.  
- Only operate the device at its intended voltage.  
- In case of repairs, only use original spare parts, to avoid serious damage.  
- It must be noted that operator or connecting errors lie outside the influence of Maximex GmbH & CO. KG and we cannot accept any liability for resulting damages.  
- Always check whether the device is suitable for the respective place where you intend using it.

### General information

1. Electric appliances, packaging material etc. should be kept away from children.
2. Dispose packaging and worn out material correctly and in an environmentally friendly way.
3. Do only pass on the device including the operation manual.
4. Subject to change of design and technical data without notice for the sake of constant product improvement.

### Guarantee

We grant a two year guarantee for this device. Guarantee includes repair of all defects that were not caused by defective material or fabrication mistakes. Guarantee does not apply on wearing parts. Since Maximex GmbH & Co. KG has no influence on correct and appropriate assemblage and operation it is obvious that guarantee applies only on completeness and proper condition. Maximex GmbH & Co. KG takes on neither liability nor guarantee for damages or consequential damages in connection with this product. This applies especially when the device was altered or repaired, when circuits were changed or non-original spare parts were used or when damage was caused by false or negligent operation or abuse.

### Environmental protection notice

At the end of its life span this product may not be disposed as normal household waste but must be disposed of at a collecting place for recycling of electrical and electronic equipment. The icon on the product, in the manual, or on the packing points to this fact. The materials are recyclable according to their labelling. Through reusing, material recycling, or other forms of utilisation of old devices you make an important contribution to the protection of the environment. Please ask the local administration for the responsible waste disposal centre.

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