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Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. SAFETY INSTRUCTIONS

- The device is intended exclusively for private and not for commercial use. Only use this device as describe in this instruction manual.
- Never try to repair it by yourself. It must be repaired by the manufacturer or its service agent or a similarly qualified person in order to avoid an electric shock.
- Keep it away from heat sources such as radiators, to avoid deformation of plastic parts.
- Do not use coarse or abrasive sponges/cloths or steel wool to clean the appliance.
- Do not attempt to service the Fingertip Pulse Oximeter. Only qualified service personnel should attempt any needed internal servicing.
- Do not use the pulse oximeter in situations where alarms are required .The device has no alarms.
- Prolonged use or the patient's condition may require changing the sensor site periodically.
- Change sensor site and check skin integrity, circulatory status and correct alignment at least every 2 hours.
- SpO2 measurements may be adversely affected in the presence of high ambient light.
- Shield the sensor area (with a surgical towel, or direct sunlight, for example) if necessary.
- The following reason will cause interference and defibrillators

- (1)High-frequency electrosurgical.
- (2)Placement of a sensor on an extremity with a blood pressure cuff arterial catheter, or intravascular line.
- (3)The patient has hypotension severe vasoconstriction severe anemia or hypothermia.
- (4)The patient is in cardiac arrest or is in shock.
- (5)Fingernail polish or false fingernails may cause inaccurate SpO2 readings.

Warnings

- WARNING: EXPLOSION HAZARD — Do not use the Oximeter in a flammable atmosphere where concentrations of flammable anesthetics or other materials may occur.
- WARNING: Do not throw batteries in fire as this may causes them to explode.
- WARNING: Do not attempt to recharge normal dry-cell batteries, they may leak, And may cause a fire or even explode.

- WARNING: Do not use the pulse oximeter in an MRI or CT environment.
- CAUTION: Keep the operating environment free of dust, vibrations, corrosive, or flammable materials, and extremes of temperature and humidity.
- CAUTION: Do not operate the unit if it is damp or wet because of condensation or spills.
- Avoid using the equipment immediately after moving it from a cold environment to a warm, humid location.
- CAUTION: Never use sharp or pointed objects to operate the front-panel switches.
- CAUTION: The battery must be taken out from the battery compartment if the device will not be used for a long time.
- CAUTION: The device shall only be used if the battery cover is closed.
- CAUTION: The battery must be proper disposed according to local regulation after their use.

B. HOW TO USE IT

1) Insert or replace batteries

- Remove the battery cover on the back of the unit.
- Insert 2 AAA batteries observing the correct polarity indicated.
- Replace the cover.

2) Operation

Turn the Pulse Oximeter on/off

Put one of fingers into rubber hole of the Oximeter (it is best to put the finger thoroughly) with nail surface upward, then releasing the clamp.

Press power button to turn the Pulse Oximeter on. The oximeter will automatically be powered off when no finger in the device for longer than 16 Seconds

Read correspondent data from display screen.

The display interface of "OLED" can rotate four directions with six different display modes after pressing the power button for less than 0.5s.It is shown as below:

Note: when battery power is at lowest level, the battery capacity indicates symbol of "in OLED, remind users of replacement of battery.

TEMPERATURE FUNCTION

When the oximeter is off, press the ON / OFF button for at least 3 seconds until a small beep sounds.

Place the oximeter in front of your forehead, making sure your fingers are not on the sensor. Wait a few seconds.

A new beep is emitted when the temperature measurement is completed. You can then read the result on the LCD screen. The display is in degrees Celsius and in degrees fahrenheit.

The display turns off automatically after 3 seconds.

C. CARE AND MAINTENANCE

*Switch off the power and take out the batteries before cleaning. Keep the exterior surface of the device clean and free of dust and dirt. Cleaning exterior surface (OLED display screen included) of the unit with a dry and soft cloth. Use 75% density of medical alcohol to clean the surface and use dry fabric with little alcohol to avoid alcohol permeates into

O 5001 / OXI1408T

the device.

CAUTION: Don't use strong solvent. For example, acetone.

CAUTION: Never use an abrasive such as steel wool or metal polish.

CAUTION: Do not allow any liquid into the product, and do not immerse any parts of the device into any liquids.

CAUTION: Avoid pouring liquids on the device while cleaning.

CAUTION: Don't remain any cleaning solution on the surface of the device.

*Replace the batteries timely when battery indication is low. Clean surface of the Pulse Oximeter before it is used in diagnosis for patients.

*Remove the batteries inside the battery cassette if the Oximeter will not be operated for a long time.

*It is better to preserve the product in a place where ambient temperature is $-10-40^{\circ}\text{C}$ and humidity is 10%-80%.

*Regular inspection to make sure that no obvious damage existed to affect the safety and performance of device.

*No flammable substance, overtop or lower temperature and humidity existed in operation conditions.

D. TROUBLESHOOTING

Table 5.2.1 troubleshooting

*Problems: Oxyhemoglobin or heart rate can not be shown normally

*Possible Reason: Finger is not plugged correctly. 2. Patient's perfusion is too low to be measured.

*Resolutions: Retry by plugging the finger. 2. Try some more times, If you can make sure about no problem existing in the product, Please go to a hospital timely for exact diagnosis

*Problems: Oxyhemoglobin or heart rate is shown unstably

Possible Reason:

1. Finger might not be plugged deep enough.

2. Finger is trembling or patient's body is in movement status

Resolutions:

1. Retry by plugging the finger

2. Try not to move, Let the patient keep calm.

*Problems: The Oximeter can not be powered on

*Possible Reason: 1. Power of batteries might be inadequate or not be there at all

2. Batteries might be installed incorrectly 3. The Oximeter might be damaged

*Resolutions: 1. Please replace batteries 2. Please reinstall the batteries 3. Please contact with local customer service center.

*Problems: The screen are suddenly off

*Possible Reason: 1. The product is automatically powered off when no signal is detected longer than 8 seconds 2. Power quantity of the batteries is exhausted.

*Resolutions: 1. Normal 2. Replace the batteries.

E. FEATURES

*Lightweight for carrying and Easy-To-Use.

*Manually adjust the direction of interface .

*Color OLED display, simultaneous display for testing value and plethysmogram.

*Low Perfusion down to 0.3%.

*Visual alarm function. Real-time spot-checks.

*Low Battery voltage indicator.

*Automatically switch off.

*Standard two AAA 1.5V Alkaline Battery support more than 20 hours continuous work.

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

○ *Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.*

○ *It is important to limit the battery consumption and to respect the following rules:*

● *Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.*

● *Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.*

○ *Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.*

○ *To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.*

○ *Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.*

○ *Replace all the battery at the same time. Never mix up alkaline battery with saline battery.*

○ *Used battery has to be removed.*

○ *Also, remove the battery if you're not using the apparel for a long time.*

○ *Never try to short-circuit the terminal of a battery.*

○ *Never throw away the battery in fire because they can explode.*

○ *Take out the accumulator from the apparel before reload it.*

○ *We recommend that an adult watch if a child change the battery to respect those rules.*

○ *If you swallow a battery, go immediately to your doctor or the hospital.*

○ *For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.*

O 5001 / OXI1408T

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE

Carton et emballage papier à
trier / Kartons und
papierverpackung recyceln /
Verpakkingsmateriaal in karton
en papier te sorteren / Recycle
carton and paper packaging