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Caro(a) Cliente

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EN INSTRUCTIONS

Dear customer,

Your new up-lifting seat offers optimum functionality and comfort. We wish you much enjoyment with it. Read these instructions carefully before assembling and using the product! Retain the instructions for future reference! If you pass the product on, please also pass on these product or non-observance of the information in these instructions!

Intended Use

The up-lifting seat is only to be used as an assistant to help persons sit down or get up from the chair. It is designed for indoor and private use. The product is unsuitable for trade use.

Technical Data:

Max. load: 105 kg
Step size: 54 x 45 mm
Weight: 3450 g

Use instruction

1. The up-lifting seat have 6 grade of weight adjustment.
2. Adjust the weight capacity according to body weight. Snap the spring into hooks.
3. Place the seat in the chair after adjusting in a correct grade. Seat will fall in suitable weight capacity when people sit on it.
4. Seat will help people stand up easier. Make sure people hold the armrest of a chair when use the seat.

Safety instruction

- Only use the up-lifting seat for the intended use.
- Only use the up-lifting seat in the chair with armrest.
- Do not hold the edge of seat when sitting down. Your hand may clamp by seat!
- Do not stretch the hand into the gap of seat when people sit down on it.
- Do not sit down when people adjust the weight capacity.
- Do not make any modifications to the product and do not repair the item yourself!
- Only obtain spare parts from the manufacturer or the wholesaler. Incorrect repairs can cause unforeseeable damage!
- Keep the packing material away from children (risk of suffocation).
- Ensure that another person is close by. This person can help you with sitting down or standing up if necessary.
- Do not use this product with visible or suspected damage!

Cleaning

CAUTION

Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface. Wipe off the seat with a damp cloth.

Disposal

Dispose of the packing in an environmentally friendly manner. Dispose of it where valuable waste is collected. Dispose of the product through an approved disposal company or the municipal disposal facilities. Observe the current regulations. If in doubt, consult your local authority about an environmentally friendly disposal

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