**Rollstuhlrampe RFM Aluminium**

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### Aluminium Wheelchair Ramp RFM

### Instruction manual

Please read this manual carefully and keep it in a safe place; it contains important information for correct and safe use. This manual is for guidance only.

### Introduction

The mobile ramps are made of aluminium and are light and easy to transport in the supplied bag. Black anti-slip pads and an approx. 4.6 cm high edge prevent the wheelchair from slipping. The ramps can be pulled out/extended for adjustment and are therefore versatile.

### Specification

Length	Usable Width/total Width	Weight	Material	Max. Load
152 cm	15 cm/19 cm	3,2 kg/pc.	Aluminium	270 kg

### Assembly

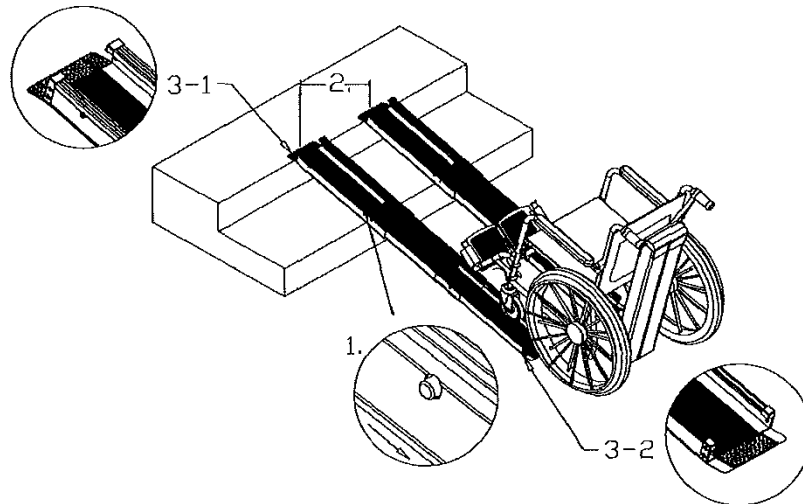
1. To extend the ramp, press the yellow knob on the outside of the ramp. The inner part of the ramp runs in a rail and can now be pulled out.
2. Push the two elements apart until they are secured in the middle by a second locking position.
3. Check that the parts are securely locked.
4. A safety pin is located at the top of each ramp to stabilize the ramps during operation. These can be used for drilling when the ramps are placed on the upper landing.
5. Place the ramps so that the wheels of the wheelchair fit in the middle

of the ramp element.

6. Make sure that the end angled upwards is at the top of the ramp, and the end angled downwards is at the bottom of the ramp.

7. Very smooth surfaces are not suitable for non-slip surfaces; make sure the ramps are securely positioned before use. In permanent position it is advisable to fasten the ramps with screws.

8. To shorten the ramp, press the yellow knob on the outside of the ramp. Keep the knob pressed and the outer part of the ramp with one hand. Now push down the inner part of the ramp in the guide until the lock is released and push it back into the starting position until it clicks into place.

#### Storage/transport (in the bag)

1. Proceed as described in point 7.
2. Place the ramps in the carrier bag - made of high-quality fabric with durable double embroidery. This can either be attached to the wheelchair by means of adjustable eyelets or transported by shoulder strap.
3. The practical cover with Velcro fastener serves for safe storage.
4. Length: approx. 90 cm, Ready: approx. 28 cm, Height 6 cm, Weight: approx. 500 g, Material: Nylon
5. To prevent the ramp parts from scratching each other, place a piece of cardboard in between.

#### Safety instructions

- Do not exceed the maximum load of 270 kg - when wheelchairs and wheelchair users are transported, the weight of the driver must be taken into account.
- We recommend using the ramps with the help of a qualified person. This should be next to the ramp.
- Always move the wheelchair backwards down the ramp, never forwards.
- Make sure that the latches are fully engaged.
- Check the product for integrity and safety before each use.
- Check the product regularly for damage or wear.

#### Cleaning

These ramps are made of anodised aluminium and are tarnish resistant. If necessary, clean the ramps with a commercially available cleaning agent, water and a dry cloth.

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